




Bedside Terminal

Quick Start Guide

UD43067B


1 Appearance

Device Appearance

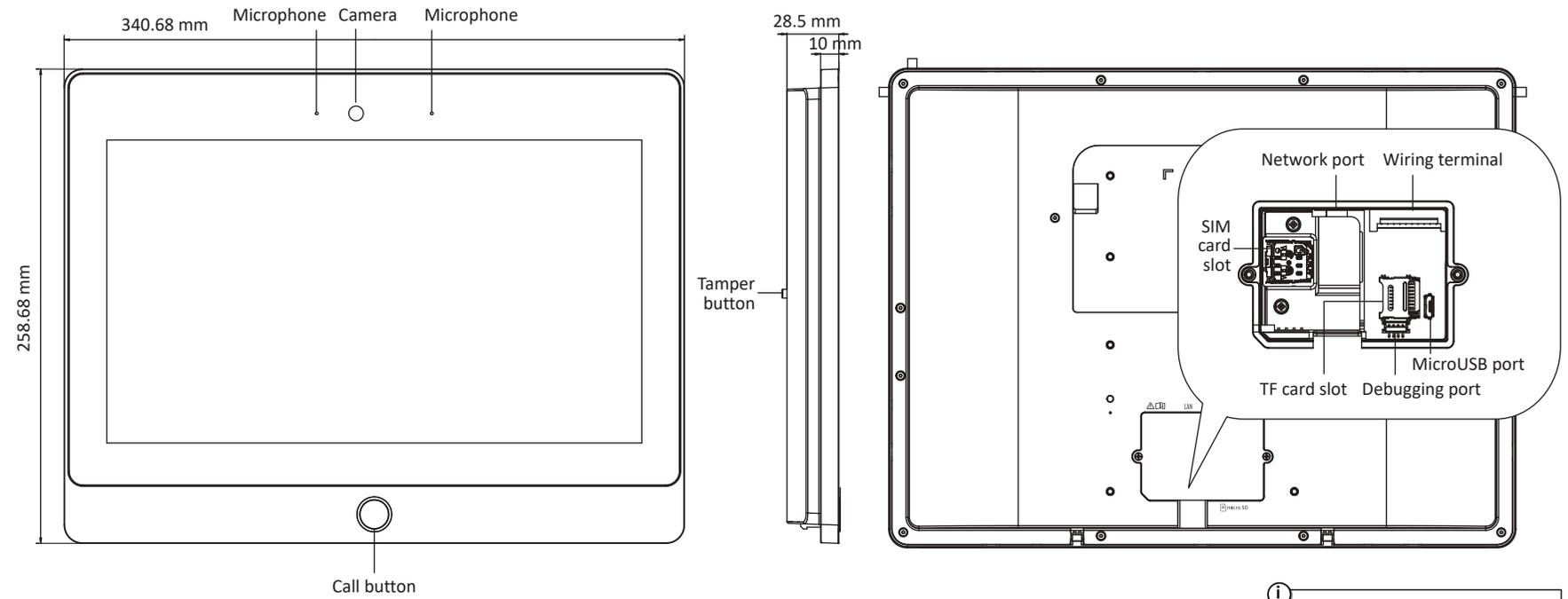

i Debug serial port for debugging purposes only.

2 Wiring

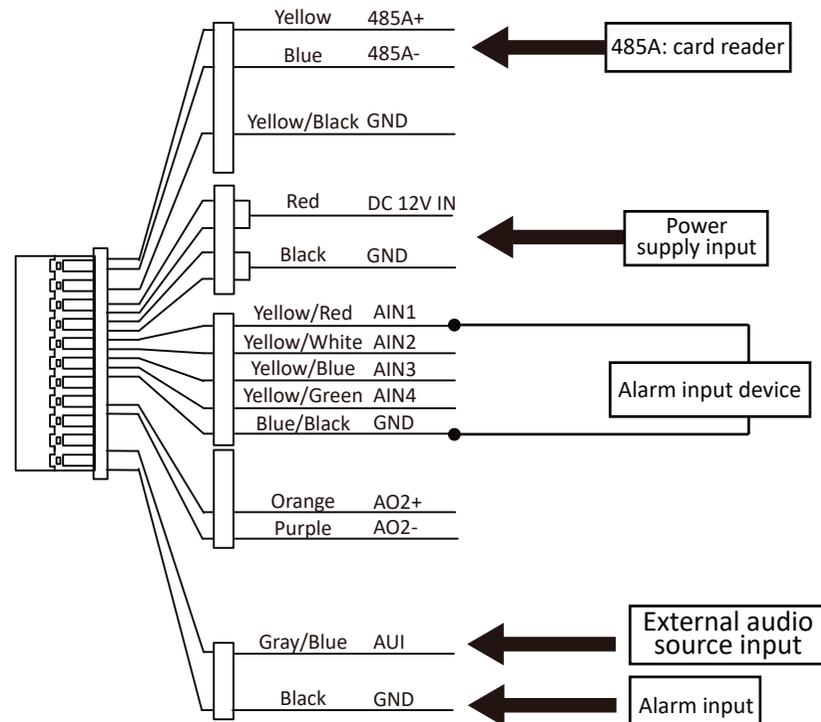

i AUI of WH version is used for connecting a medical handle.

About this Document

- This Document includes instructions for using and managing the Product. Pictures, charts, images and all other information hereinafter are for description and explanation only.
- The information contained in the Document is subject to change, without notice, due to firmware updates or other reasons. Please find the latest version of the Document at the Hikvision website (<https://www.hikvision.com>). Unless otherwise agreed, Hangzhou Hikvision Digital Technology Co., Ltd. or its affiliates (hereinafter referred to as "Hikvision") makes no warranties, express or implied.
- Please use the Document with the guidance and assistance of professionals trained in supporting the Product.

About this Product

This product can only enjoy the after-sales service support in the country or region where the purchase is made.

Acknowledgment of Intellectual Property Rights


- Hikvision owns the copyrights and/or patents related to the technology embodied in the Products described in this Document, which may include licenses obtained from third parties.
- Any part of the Document, including text, pictures, graphics, etc., belongs to Hikvision. No part of this Document may be excerpted, copied, translated, or modified in whole or in part by any means without written permission.
- **HIKVISION** and other Hikvision's trademarks and logos are the properties of Hikvision in various jurisdictions.
- Other trademarks and logos mentioned are the properties of their respective owners.

LEGAL DISCLAIMER

- TO THE MAXIMUM EXTENT PERMITTED BY APPLICABLE LAW, THIS DOCUMENT AND THE PRODUCT DESCRIBED, WITH ITS HARDWARE, SOFTWARE AND FIRMWARE, ARE PROVIDED "AS IS" AND "WITH ALL FAULTS AND ERRORS". HIKVISION MAKES NO WARRANTIES, EXPRESS OR IMPLIED, INCLUDING WITHOUT LIMITATION, MERCHANTABILITY, SATISFACTORY QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE. THE USE OF THE PRODUCT BY YOU IS AT YOUR OWN RISK. IN NO EVENT WILL HIKVISION BE LIABLE TO YOU FOR ANY SPECIAL, CONSEQUENTIAL, INCIDENTAL, OR INDIRECT DAMAGES, INCLUDING, AMONG OTHERS, DAMAGES FOR LOSS OF BUSINESS PROFITS, BUSINESS INTERRUPTION, OR LOSS OF DATA, CORRUPTION OF SYSTEMS, OR LOSS OF DOCUMENTATION, WHETHER BASED ON BREACH OF CONTRACT, TORT (INCLUDING NEGLIGENCE), PRODUCT LIABILITY, OR OTHERWISE, IN CONNECTION WITH THE USE OF THE PRODUCT, EVEN IF HIKVISION HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES OR LOSS.
- YOU ACKNOWLEDGE THAT THE NATURE OF THE INTERNET PROVIDES FOR INHERENT SECURITY RISKS, AND HIKVISION SHALL NOT TAKE ANY RESPONSIBILITIES FOR ABNORMAL OPERATION, PRIVACY LEAKAGE OR OTHER DAMAGES RESULTING FROM CYBER-ATTACK, HACKER ATTACK, VIRUS INFECTION, OR OTHER INTERNET SECURITY RISKS; HOWEVER, HIKVISION WILL PROVIDE TIMELY TECHNICAL SUPPORT IF REQUIRED.
- YOU AGREE TO USE THIS PRODUCT IN COMPLIANCE WITH ALL APPLICABLE LAWS, AND YOU ARE SOLELY RESPONSIBLE FOR ENSURING THAT YOUR USE CONFORMS TO THE APPLICABLE LAW, ESPECIALLY YOU ARE RESPONSIBLE, FOR USING THIS PRODUCT IN A MANNER THAT DOES NOT INFRINGE ON THE RIGHTS OF THIRD PARTIES, INCLUDING WITHOUT LIMITATION, RIGHTS OF PUBLICITY, INTELLECTUAL PROPERTY RIGHTS, OR DATA PROTECTION AND OTHER PRIVACY RIGHTS. YOU SHALL NOT USE THIS PRODUCT FOR ANY PROHIBITED END-USES, INCLUDING THE DEVELOPMENT OR PRODUCTION OF WEAPONS OF MASS DESTRUCTION, THE DEVELOPMENT OR PRODUCTION OF CHEMICAL OR BIOLOGICAL WEAPONS, ANY ACTIVITIES IN THE CONTEXT RELATED TO ANY NUCLEAR EXPLOSIVE OR UNSAFE NUCLEAR FUEL-CYCLE, OR IN SUPPORT OF HUMAN RIGHTS ABUSES.
- IN THE EVENT OF ANY CONFLICTS BETWEEN THIS DOCUMENT AND THE APPLICABLE LAW, THE LATTER PREVAILS.

3 Installation

Arm Mounting

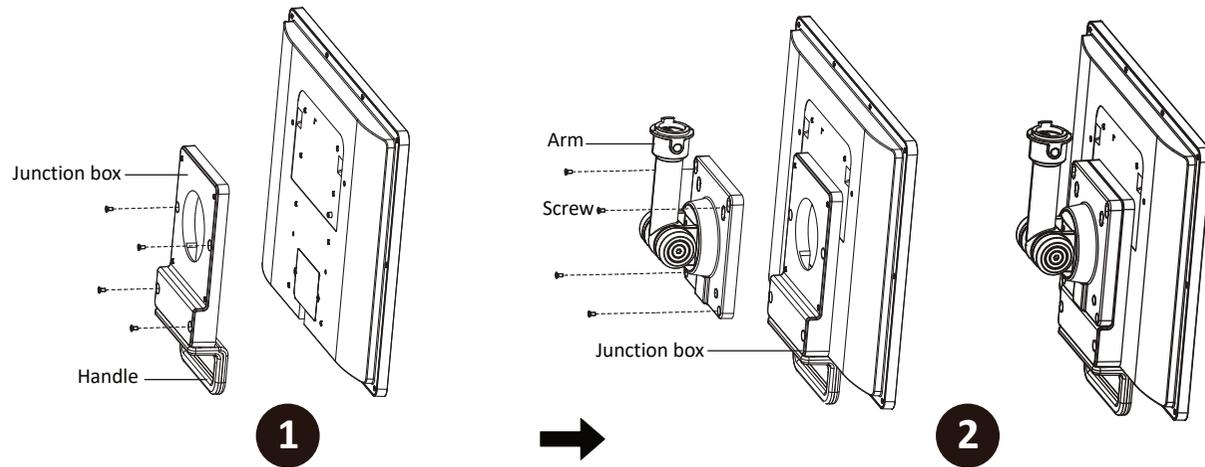

- 1 Route the reserved cable through the junction box and secure the junction box in place by aligning with the positioning hole on the rear panel of the device.

Secure the junction box to the device using four screws.

- 2 Route the reserved cable through the arm bracket, align it with the junction box, and secure the arm bracket to the junction box using four screws.

4 Quick Setup

You must activate the device when you use it for the first time. You will be automatically directed to the activation interface when the device is powered on.

Local Activation

1. After the device is powered on, the screen turns on and you will be automatically directed to the activation interface.
2. Create your activation password and confirm the password.
3. Tap "OK." The device will automatically go to the Quick Setup interface.



In order to improve security while using the product on the network, set a password between 8–16 characters in length, consisting of at least two kinds of the following characters: digits, lowercase and uppercase letters, and special characters.

Quick Setup

Upon activation, the device will automatically go to the Quick Setup interface.

1. Set security questions, select three different security questions, and enter the answers. Then tap "Next."
2. Select the work mode as "Visitor Terminal" or "Ward Terminal", and tap "Next."
3. Configure the network settings and tap "Next."

- Wired network mode: You can manually enter the IP address, subnet mask, and default gateway, or when you enable "Obtain an IP address automatically," the device will automatically obtain network parameters.

- Wireless network mode: Select a Wi-Fi from the Wi-Fi list.

Note: The device uses the wired network mode when a network cable is connected to the device.

4. Configure session parameters, create an SIP registration password, and tap "Finish."



For other configurations, see User Manual.

Warning

- While installing and using this device, all electrical safety regulations of the country and region where the product is used must be strictly observed.
- Warning! Always replace a fuse with one that has the same kind and rating. Otherwise there is a risk of electric shock or fire.
- Caution! This device can only be used with supporting brackets. Using with others (such as carts, shelves, or handling devices) may lead to instability and personal injury.
- To prevent possible hearing damage, do not listen to music at high volume for an extended period of time.
- A centralized power supply, if used, should meet the power requirements for the device. When connecting wires, disassembling, or similar, always cut off the power supply to the device. Never conduct such tasks with the power connected. Maintain proper ventilation around the device to avoid heat buildup.
- If the device emits smoke, odor, or noise, immediately disconnect the device from the power supply, remove the power cable, and contact the distributor or service center.
- If the device is not working properly, contact the store where you purchased the device or the nearest service center. Do not disassemble or modify the device in any way. (The company will not assume responsibility for any issues that arise due to unauthorized modification or repair).

Caution

- + indicates the positive polarity. - indicates the negative polarity.
- Do not place an exposed flame source, for example, a lit candle, on the device. The USB port on the device can only be used for connecting a USB flash drive. Do not use it for charging or carrying other loads. The serial port on the device is only used for debugging and is not intended for users.
- The metal part of the fingerprint sensor on this device is hot. Do not touch the metal part in half an hour after power off.
- Please install the device in strict accordance with the installation instructions set forth in this guide. For safety, the device must be securely mounted to a wall or ceiling.
- Do not drop objects on the device or shake the device vigorously, and keep the device away from magnetic interference. Avoid installing the device in a place where the surface vibrates or where the device is vulnerable to impact. Otherwise, the device may be damaged.

- Please do not use the device in high temperature, low temperature, or high humidity environments. For specific temperature and humidity requirements, refer to the parameter table. Devices for indoor use must not be exposed to rain or high humidity.
- Do not expose the device to direct sunlight or place it in poorly ventilated areas or near heat sources such as heaters or heating source. Failure to do this may lead to a fire hazard.
- Network security issues may arise when the device is connected to the internet. Please enhance your personal information and data security protection. When you discover potential network security risks, please contact us immediately.
- Please understand that you are responsible for the configurations, all passwords, and relevant security issues of the product. You are also responsible for keeping your user name and password safe.
- Keep all of the original packaging materials of the device safe, so that it can be used to re-package the device to be sent back to the distributor or manufacturer should any issues arise. The company assumes no responsibility for accidental damage caused by the use of non-original packaging materials during transportation.
- Biometric identification products may not be fully adapted to any anti-counterfeiting environment. Please combine authentication methods for high-security places.
- It should be noted that biometric identification may not be fully accurate due to the quality of the captured photo and the effect of ambient light.

Instruction

Installation and maintenance personnel requirements:

- Have basic knowledge and installation skills for access control systems and compositions.
- Have basic knowledge and operation skills for low voltage wiring and low voltage electric circuit wiring.
- Have basic knowledge and skills regarding basic network security and be able to read and understand the content of this manual.



Please scan the QR code to get the User Manual